medi

medi Butler

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medi Butler

General Instructions

Please read through the entire instructions for use carefully.

They contain important information concerning the product itself, as well as its handling, maintenance and disposal.

Keep the instructions for use. You may wish to read them again at a later point in time.

If you have any questions, please contact the medical supply store or the medi Consumer Service.

Compression therapy aids include, among other items, medical compression stockings which are manufactured in different lengths, compression classes and with different knitting methods. Compression stockings are used for a variety of purposes, including* treating venous disorders and conditions and for treating oedema caused by lymphoedema and lipoedema.

¹ Donning and doffing aids protect and preserve the stocking material and make it easier to put on and take off medical compression stockings. In particular, they also enable patients with limited mobility to use the products by reducing the amount of motion and force needed to put them on and take them off.

* For indications for medical compression stockings see medi and/or mediven instructions for use.

Purpose and use

Donning aids for medical/clinical compression socks and stockings

Indications for donning and doffing aids include

- Hemiplegia
- Loss of power due to age
- Osteoarthritis/rheumatism
- Morbid obesity
- Extensive spinal/hip/knee stiffness
- Degenerative condition of the hands/related to the hands
- Sequellae from injuries/amputations

¹ S2k guideline: Medical compression therapy of the extremities using medical compression stockings (MCS), venous compression bandage (VCB) and medical adaptive compression systems (MACS). Published online at: <https://www.awmf.org/leitlinien/detail/II/037-005.html>
Chapter: 2.1.3 Donning and Doffing Aids

Contraindications

No contraindications are known.

Important general instructions

To prevent injuries and damage to the delicate knitted fabric, please make sure your finger and toenails are well-kempt and the skin on your heels is

smooth.

Please dispose of this product as soon as it shows signs of wear or damage.

Storage instructions

Please store the medi Butler in a dry place and keep it out of direct sunlight.

Use

Adjusting handle length for the medi Butler with variable handles

First, please adjust the handle so that it is at the right length for you.

To do this, loosen the four hexagon screws with the enclosed spanner, until the length of the metal frame can easily be adjusted.

To find the right handle length for you, sit on a chair or the edge of the bed and put the medi Butler with variable handles on the floor in front of you.

Now reach for a handle and pull it out of the frame until you can reach it comfortably and easily

Fix this height immediately by retightening the screws using the spanner.

Please repeat this procedure for the second handle on the other side.

How do you put on your compression stocking using the medi Butler?

First, place the medi Butler on a firm base.

The wider side of the base of the Butler should be closest to you and the opening of the semicircular shaft should be facing forwards.

Place the compression stocking in the step-in frame. The stocking toe points away from you here.

Open out the end of the stocking and roll it over the shaft (Fig. 1).

Pull the stocking down over the step-in frame until the toe appears at the edge of the frame (Fig. 2).

Tip: Use the medi fabric gloves or rubber gloves for this

Now place the medi Butler on the floor.

Slide your leg into the toe of the stocking, leading with your toe.

For stability, it is recommended that you support yourself using a sturdy object (e.g. table, wall) (Fig. 3).

Now you can “step into” the stable brace, foot first.

Briefly check again that the toe and ankle of the stocking are positioned correctly (Fig. 4).

Pull the brace back and forth repeatedly using both handles to pull the stocking up to your knee easily (Fig. 5).

When you reach under the knee, tilt the medi Butler backwards to remove it and set it aside (Fig. 6).

Finally, pull the compression stocking up evenly.

Please ensure that the stocking is pulled up gradually, without wrinkling or tugging it (Fig. 7).

Tip: The medi fabric gloves and rubber gloves enable you to arrange and position compressive aids on arms and legs correctly.

Cleaning the medi Butler

If desired, you can clean your donning aid with the standard commercially available disinfectants or detergents.

Please wipe down the frame and make sure that no liquid gets into the hollow part of the base (5).

Material composition

Steel

Finishing: powder-coated, white, glossy

Disposal

Please dispose of the medi donning aid correctly at the respective collection point for scrap metal.

There are no specific criteria for disposal as hazardous waste.

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**These instructions apply to the
following Butler models**

medi Butler with variable handles
medi Butler with long handles
medi Butler for panties
medi Big (XXL) Butler
medi Export Butler with long handles

Tachezy Sanit Srl.
Via Druso 48/a
39100 Bolzano
Italy
T: +39 0471 288 068
F: +39 0471 282 773
info@tachezysanit.com
www.tachezysanit.com

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
Vestheimvegen 35
4250 Kopervik
Norway
T: +47 52 84 45 00
F: +47 52 84 45 09
post@medinorway.no
www.medinorway.no

medi Polska Sp. z o.o.
ul.Łabędzka 22
44-121 Gliwice
Poland
T: +48-32 230 60 21
F: +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova Street 17
117342 Moscow
Russia
T +7-495 374 04 56
F +7-495 374 04 56
info@medirus.ru
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
Room 1703, T1, CES WEST BUND CENTER,
No. 277, Long Lan Road
Xuhui District
200232 Shanghai
People's Republic of China
T: +86-21 50582319
F: +86-21 50582319

medi Sweden AB
Box 6034
192 06 Sollentuna
Sweden
T +46 8 96 97 98
F +46 8 626 68 70
info@medi.se
www.medi.se

Medi Turk Ortopedi Medikal İc ve Dis Tic. AS
Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5
06510 Çankaya Ankara
Turkey
T: +90 312 435 20 26
F: +90 312 434 17 67
info@medi-turk.com
www.medi-turk.com

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
Great Britain
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Evhena Sverstiuka str, 11.
Kiev 02002
Ukraine
T: +380 44 591 11 63
F: +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1-336 4 49 44 40
F +1-888 5 70 45 54
info@mediusa.com
www.mediusa.com

medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-57
info@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61 2 9890 8696
F +61 2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43-512 57 95 15
F +43-512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium NV
Staatsbaan 77/0099
3945 Ham
Belgium
T: + 32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Canada Inc / médi Canada Inc
597, Rue Duvernay, Verchères
QC Canada J0L 2R0
T: +1 450-583-3317 / +1 800 361 3153
F: +1 888-583-6827
service@medicanada.ca
www.medicanada.ca

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T: +39-051 6 13 24 84
F: +39-051 6 13 29 56
info@medi-italia.it
www.medi-italia.it

medi Japan K.K.
5-1 Nihonbashi Kabutocho Chuoku
103-0026 Tokyo
Japan
T: +81 3 6778 2590
F: +81 3 5847 7901
info@medi-japan.jp
www.medi-japan.co.jp