Afterwards please tighten the 3 screws again and put the handle onto the edge of the bathtub.

Make sure that the upper plate is solidly seated on the rim of the bathtub.

Now fasten the adjusting knob till the movable Pad-1 perfectly fits at the inner surface of the bathtub. Tighten the adjusting knob as strong till HUGO cannot be moved anymore and is trapped completely (Picture 3).

Picture 3

Bathtub without 100% flat outer surface at the edge of the tub

In that case use the included self-adhesive plates to close the gap to ensure that the outer rubber-pad gets a maximum contact surface, too. Therefore remove the protective foil of a plate and stick it onto the place where the gap exists. If one plate is not enough, use plate 2 and if necessary plate 3.

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Mounting of HUGO at the bathtub


IMPORTANT: First inspect the surface where you are going to fasten the handle. Take care that the surface is completely dry and free of soap or other slippery substances!

To find the right position for the handle please sit down into the dry bathtub to check the reaching area of your hands and at which position the handle supports you the best.

Now loosen the adjusting knob till Pad-1 is in a position that HUGO fits completely onto the edge of your bathtub.


If the distance between the two pads is still not wide enough, loosen the upper 3 screws with the small Allen wrench and pull out the metal plate as far as necessary so that the handle fits completely onto the edge.
(Picture 2.)

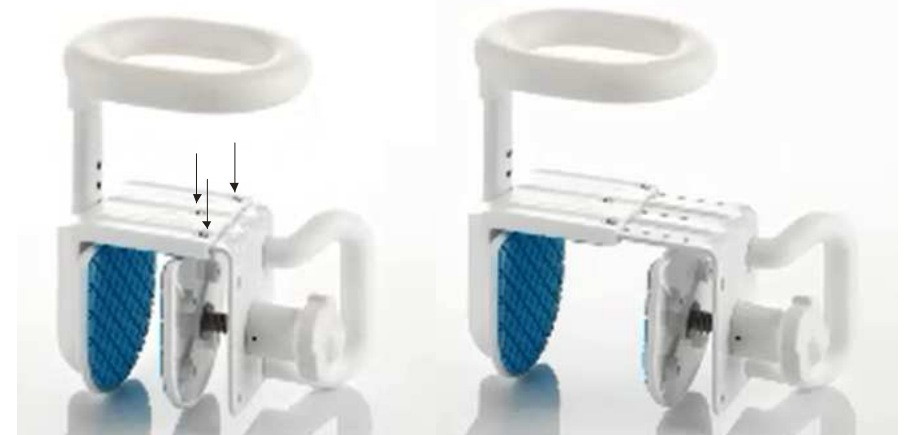Picture 2

This grip can be fixed on the left or right side of the metal plate!
It depends on your position in the bathtub and where you need the grip.

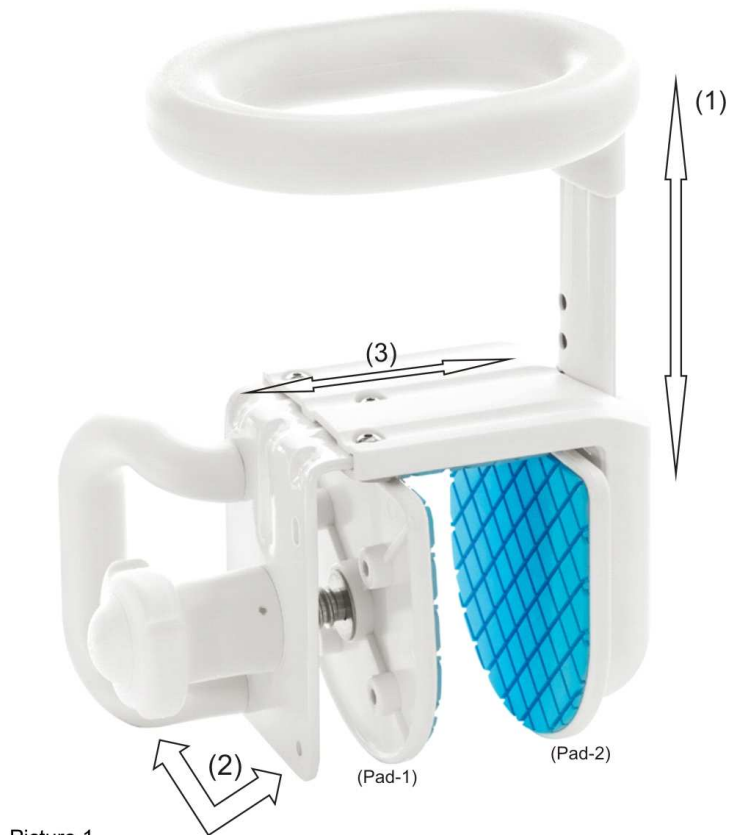HUGO is now ready for mounting onto the bathtub

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Implementation - Using instructions

The bathtub handle HUGO is essential for persons who have difficulties in controlling their balance. HUGO offers you safety and best comfort in your daily living. With the possibilities in the height and width adjustment the bathtub handle can be mounted optimally onto the edge of your bathtub. Fix the bathtub handle onto the edge of your bathtub.

Scope of delivery-set

- 1x clamping device with blue rubber-pads
- 1x main handle with tube for height adjustment
- 1x small, white side-grip
- 1x welded bag with: 2 Allen wrench / 2 short, 2 long screws
4 washers / 2 white cover plugs
- 1x welded bag with: 3 self-adhesive spacer plates

Assembly (Picture 1)

1. Please insert the *main handle with tube* into the upper hole of the clamping device as shown in Picture 1 (1). Adjust the handle to the desired height and use the big Allen wrench to tighten both long screws with washer.

Afterwards insert the white cover plugs into the screw holes.

2. Mounting of the white side-grip: First loosen Pad-1 with the white adjusting knob till the 2 smaller screws and washer can be inserted from inside through the holes in the metal plate to fix the side grip as shown in picture 1(2). Tighten the screws by using the small Allen wrench. The grip can be mounted at the desired side (left or right) to provide best support.

Technical data:

Maximum capacity:	120 kg vertical and horizontal
Weight of the product:	2.5 kg
Clamping width:	adjustable 2.9 to 13.5 cm
Height of the handle:	adjustable 8.0-14.0 cm
Material:	Plastic, stainless steel, non-skid rubber

Precautions, service and maintenance

- Inspect all parts of the bathtub handle for proper condition and completeness before mounting and using the handle the first time.
- In any circumstance pay attention to the maximum capacity of 120 kg.
- Control constantly the mechanical state of the product. If parts are defective, worn or damaged, they have immediately replaced by an authorized person.
- Clean your product continuously with a damp sponge, gentle soap and a household detergent/disinfectant. Wipe the handle with a dry soft cloth.
- Do not use a brush nor an abrasive detergent or abrasive powder. As to the disinfection in a hospital or similar establishment we refer to the responsible for hygiene in these establishments.

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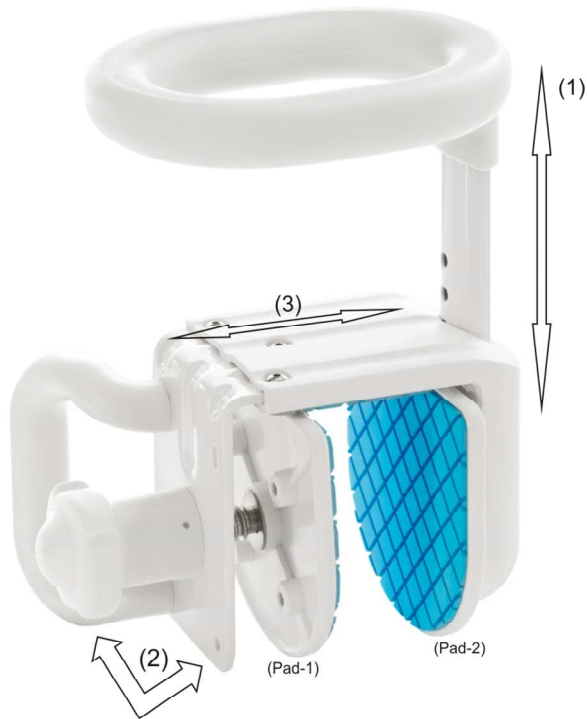
Bild 1

Instructions:

Please make sure, that all instructions were entirely understood, before mounting and using this product the first time. If there are problems to understand the notes, safety instructions, using instructions or assembly, please ask your dealer for further information to avoid any injury or material damage. Consult with your physician or therapist to ensure that this product will be suitable for you and will match your requirements. Don't use this product, before this advice or a training by a specialist was conducted.

Warranty

rehasstage GmbH offers you a 2-year warranty on this product against all defects or faults in manufacturing or materials. The warranty does not cover damage resulting from misuse, dismantling, or repair by unauthorised persons or damage resulting from non-observance of the using instructions or accidents caused by false cleaning or caused by commercial use. Standard repairs or exchanges during the warranty period will not affect the expiry date of this warranty. This guarantee in no way affects your rights under statutory law. Keep your invoice, because it confirms your purchase.

Thank you for purchasing the HUGO Bathtub Handle. This product will accompany you through your daily living. Approved and useful it is of assistance to you because HUGO is so easy to install and provides best support.

Our products were designed and produced in accordance with the existing quality standards and safety regulations and in addition according to the provision of the Council Directive 93/42/EEC for Medical Devices.

IMPORTANT: Please read the following instructions for your safety and to learn of the features of this product. Please do not use the product before you completely read these instructions. Keep this manual so that the information can be consulted again later.

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