# DUO MOTION / DUO MOTION 24

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## Table of contents

|                                                  |    |
|--------------------------------------------------|----|
| Preface .....                                    | 9  |
| Declaration of conformity .....                  | 9  |
| Service life .....                               | 9  |
| Disposal .....                                   | 9  |
| Identification plate .....                       | 9  |
| Reuse .....                                      | 9  |
| Warranty .....                                   | 9  |
| Parts of the product .....                       | 10 |
| Indication .....                                 | 11 |
| Scope of delivery .....                          | 11 |
| Safety instructions .....                        | 11 |
| Assembly .....                                   | 11 |
| Assembly and disassembly of the 24" wheels ..... | 12 |
| Double-function parking brake .....              | 12 |
| Height adjustment of the footrests .....         | 12 |
| Cleaning   Maintenance .....                     | 12 |
| Technical data .....                             | 13 |

## Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

## Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

## Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

## Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

## Identification plate

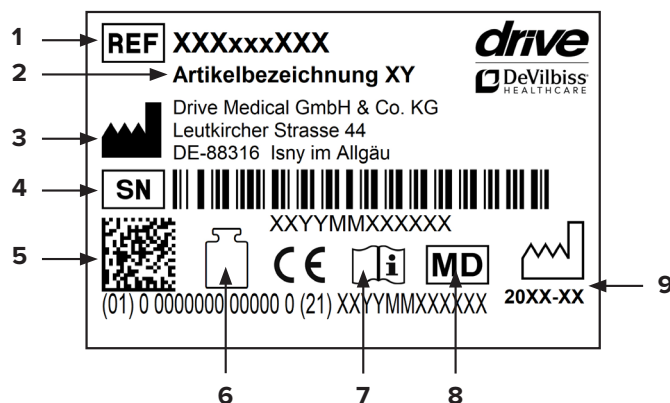1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

## Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at [www.drivedevilbiss.de](http://www.drivedevilbiss.de)

## Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

### EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes, armrests and handles

## Parts of the product

## Duo Motion / Duo Motion 24

- |                |                               |                                    |
|----------------|-------------------------------|------------------------------------|
| 1. Push handle | 5. Seat section               | 9. Footrests                       |
| 2. Backrest    | 6. Toilet bucket with cover   | 10. Double-function parking brake  |
| 3. Armrest     | 7. Footrest locking mechanism | 11. Brake for 24" wheel (optional) |
| 4. Cover plate | 8. Heel strap                 | 12. 24" wheel (optional)           |

## Indication

### INDICATION:

Significant to full impairment of mobility/walking/transfer in the case of structural and/or functional damage to the lower extremities (including amputation, after-effects of injuries, musculoskeletal/neuromusculoskeletal movement disorders) and Inability to move a wheelchair (for example, in the case of limited strength and grip function or a lack of coordination ability of the arms/hands).

For use of the toilet and shower (the shower must be designed such that it can accommodate a shower wheelchair).

## Scope of delivery

|     |                           |
|-----|---------------------------|
| 1 x | Base frame                |
| 1 x | Back unit                 |
| 4 x | 5" wheel (for 530500200)  |
| 2 x | 5" wheel (for 530500100)  |
| 1 x | Seat                      |
| 1 x | Hygiene cover             |
| 2 x | Star screw                |
| 1 x | Bucket with cover         |
| 2 x | Footrests                 |
| 2 x | 24" wheel (for 530500100) |
| 1 x | Tools                     |
| 1 x | Instructions for use      |

## Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down, standing up, inserting the toilet bucket, as well as when raising the armrests and locking them into place.
- If you need to move over thresholds, do so slowly and carefully, otherwise the wheels can become damaged!
- When using a wheeled commode, always use the parking brakes.
- During transfer into/out of the wheeled commode, the footrests must be swiveled out of the way.
- Never use the footrests as assistance when getting up – risk of tipping!
- Prevent unauthorized use, for example, by children.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.

- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

## Assembly

Insert the four 5" wheels into the mounting tube on the frame in the desired position and secure the wheels with the pin clips provided (**pic. 1-2**).

Place the preassembled back unit on the base frame and secure it by turning the star screws clockwise (**pic. 3**).



Clip the seat section to the frame (**pic. 4**).

Put the backrest cover over the armrests and close it as shown. Push the bucket from the rear into the guide rail underneath the seat section; to remove after use, pull out backwards (**pic. 5**).

Hang the footrests provided as shown and swivel them inwards until they audibly click into place (**pic. 6**).

To release the footrests, press the lever inwards and swivel the footrests to the side (**pic. 7**).

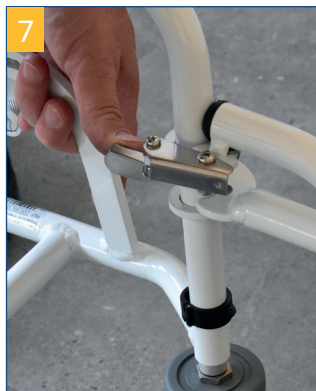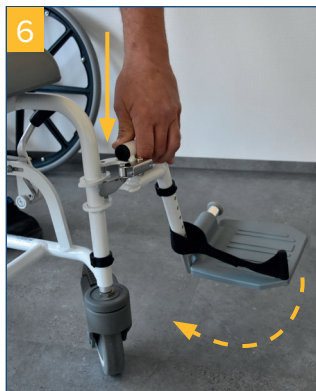
Put the backrest cover over the armrests and close it as shown (**pic. 8**).

## Assembly and disassembly of the 24" wheels

„Assemble the brakes as shown on both sides, allowing a distance of approx. 1 cm to the mounted 24" wheels. The 24" wheels are equipped with a quick-release axle and thus easy to install. The tool needed for this is enclosed.“ (**pic. 9**).

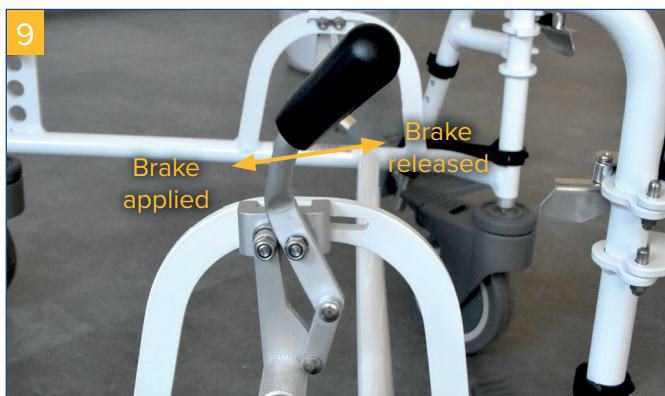## Double-function parking brake

To apply the brake, press the lever downwards (**pic. 10**). At the same time, the directional position of the wheel is blocked.

To release the brake, press the head of the lever downwards (**pic. 11**).

## Height adjustment of the footrests

Pull up the footrest slightly. Pull out the clip and put it in again at the required position (**pic. 12**).

Pull out the footrest slightly further if necessary. Set the other footrest to the same height as described.

**IMPORTANT: Never use footrests to assist with standing up!**

## Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillo AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.
- Check the condition of the tires regularly. They are „fail-safe“, meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage. In the event of defects, contact your specialist dealer from whom you purchased this product.

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Drive Medical GmbH & Co. KG