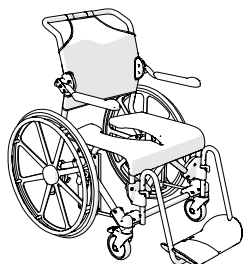
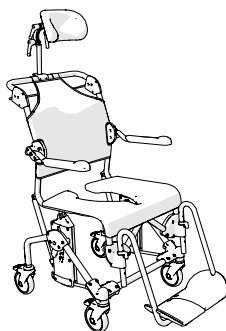
Swift Mobil-2

Swift Mobil 24"-2

Swift Mobil Tilt-2

Swift Mobil-2 XL

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Thank you for choosing an Etac product.

In order to avoid injury during handling and use, please read through this manual first.

In the manual, the user is the person sitting in the chair. The assistant is the person operating the chair.

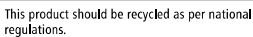
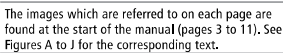
Intended use:

The Swift Mobil-2 family is a range of mobile shower and toilet chairs intended for use when carrying out hygiene care for people with decreased mobility. The shower chair can be used in showers, at sinks or over the toilet, as well as for transfers to and from the hygiene room.

Contra indications: Clinical experience shows that there are no harmful effects of using Swift Mobil-2.

Swift Mobil-2 has been tested by a testing institute and complies with the requirements stated in EN ISO 10993-1 and SS EN 12182.

This symbol appears in the manual along with text. It indicates instances where the safety of the user or the assistant may be at risk.

Item list: ● = Applies to, ○ = Accessories

See the item list on the right which displays the information applicable to each product.

A1. Bolt, A2. Leg support, A3. Seat, A4. Arm support, A5. Back support upholstery

A6. Head support

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com. Service life 7 years. For complete information regarding the service life of the product, see www.etac.com.

D1. Clean the product with normal non-abrasive cleaning agents (pH value of between 5-9). Rinse and dry. Disinfect the product with an alcohol-based disinfectant, 70%.
The product is otherwise maintenance-free.

D2. The product and its accessories can be safely cleaned at 85°C for 3 minutes (unless otherwise stated). This applies for reconditioning.

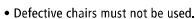
The product can be reconditioned. Refer to the reconditioning instructions at www.etac.com.

Constituent materials are resistant to common disinfectants.

Methods of surface treatment:

Lacquered surfaces = Polyester powder coating

Non-lacquered steel surfaces = Fe/Zn-Ni

E1. Make sure the castors are locked when users sit down on or get up from the chair.

- An anti-tip attachment must be used if the user is an amputee.

E2. Never stand on the foot supports as there is a risk of tipping!

E3. The sitting position has an impact on stability.
E4. Chair stability is improved if the castors point outward.
E5. See the maximum user weight for your commode chair in the table (page 5).

E6. Risk of crushing with arm supports.

E7. Do not use defective products.

Over a threshold:

F1. Chair only, put one foot on the wheel.

F2. With user, pull the chair backwards.

In stationary position:

F3. Always lock as many castors as possible when the chair is to be used in a stationary position.

F4. Out-turned castors improve the chair's stability.

Transferring:

Assess the risks and take notes. You as a carer are responsible for the safety of the user. Aids should be used if the transfer is deemed as risky. Visit www.etcac.com for manual transfer aids.

F3-F4. It is important to ensure that the castors are locked and pointing outward before carrying out transfers to and from Swift Mobil-2.

F5-F7. Fold up the arm support on the transfer side of the chair. Carry out the transfer.

English

Adjustment

See Figure G

Height

G1-G7. The height is easily adjusted by inserting the bolt in the required position. See distances in the technical data section.

Check that both sides have been adjusted to the same height as otherwise stability can be reduced. **G7.** Do not use the chair if the bolt (orange) is not locked.

Arm supports

G8. The height of the arm supports and/or width between the arm supports can be adjusted. Loosen the screws, twist the support to the desired position and retighten the screws. Then loosen and refit the arm support plate in the correct position.

Tilt angle

G9. Fit the gas piston in position A to tilt the seat at angles from -5° to 30°

G10. Fit the gas piston in position B to tilt the seat at angles from 0° to 35°

User brake

G11. When adjusting the brake, the brake pin should be set 18 to 22 mm from the rear wheel for good grip.

Leg support

See Figure H

H1. The assistant can easily raise and swivel the leg supports to the side for the user to sit down in or get up out of the chair. The foot supports are detachable.

H2. Various height adjustment options are available by moving the locking pin. The leg supports have 6 different height settings.

Tilt function

See Figure I

I1. The assistant can adjust the sitting angle with ease by holding in the handle and tilting to the required position. Make sure that the wheels are locked if the tilt function is used when the user is sitting in the chair.

Accessories

See Figure J

In the section "Accessories", you can see all accessory options available for your product. Variations may occur in different markets. There are restrictions on accessory combinations, for more information, see www.etac.com.

See Figure J	Sw M.-2	Sw M.-2 XL	Sw M.-24"-2	Sw M.-Tilt-2	See Figure J	Sw M.-2	Sw M.-2 XL	Sw M.-24"-2	Sw M.-Tilt-2
J1. Leg and calf rest, angle-variable	☐	☐	☐	☐	J21. Seat cover	☐	☐	☐	☐
J2. Footrest	☐	☐	☐	☐	J22. Soft back pad	☐	☐	☐	☐
J3. Angle adj. footrest	☐	☐	☐	☐	J23. Comfort padding	☐	☐	☐	☐
J4. Amputee legrest	☐	☐	☐	☐	J24. Armrest cushion, pair	☐	☐	☐	☐
J5. Commode pan	☐	☐	☐	☐	J25. Trunk support	☐	☐	☐	☐
J6. Pan-holder attachments	☐	☐	☐	☐	J26. Chest belt	☐	☐	☐	☐
J7. Attachments for low pan-holder mounting	☐	☐	☐	☐	J26. Positioning belt	☐	☐	☐	☐
J8. Commode pan holders	☐	☐	☐	☐	J27. Crossbar 605 mm	☐	☐	☐	☐
J9. Bucket holders	☐	☐	☐	☐	J28. Crossbar 665 mm	☐	☐	☐	☐
J10. Pan	☐	☐	☐	☐	J29. Anti-tip stabilisers including bracket	☐	☐	☐	☐
J11. Lid with handle	☐	☐	☐	☐	J30. Anti-tip stabilisers excluding bracket	☐	☐	☐	☐
J12. Splash guard	☐	☐	☐	☐	J31. Tilter	☐	☐	☐	☐
J13. Soft Comfort seat, 2 cm high, open at back	☐	☐	☐	☐	J32. Rear wheel kit	☐	☐	☒	☐
J14. Soft Comfort seat, 4 cm high, open at back	☐	☐	☐	☐	J33. Headrest and bracket	☐	☐	☐	☒
J15. Comfort seat	☐	☐	☐	☐	J34. Heel straps	☐	☐	☐	☒
J16. Soft Comfort seat	☐	☐	☐	☐	J35. Hemiplegic armrest	☐	☐	☐	☐
J17. Soft Comfort seat - narrow opening	☐	☐	☐	☐	J36. Armrest lock	☐	☐	☐	☐
J18. ROHO pressure-relieving cushion	☐	☐	☐	☐	J37. Extended back support	☐	☐	☐	☐
J19. Soft cushion - standard opening	☐	☐	☐	☐	J38. Short leg support	☐	☐	☐	☐
J20. Soft cushion - oval opening	☐	☐	☐	☐					

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Etac Supply Center AB Långgatan 12 SE-334 24 Anderstorp