Invacare® H171, H172 Stileo

- EN Suction grab bar**
User Manual
- DE Saugerhandgriff**
Gebrauchsanweisung
- FR Barre d'appui**
Mode d'emploi
- IT Maniglione di sostegno**
Istruzioni per l'uso
- ES Barra de apoyo con ventosa**
Manual de instrucciones
- NL Handgreep met zuignappen**
Gebruiksaanwijzing
- DA Håndgreb med sugeskop**
Brugsanvisning
- FI Imukuppitankoa**
Käyttöohje
- NO Gripehåndtaket med sugeskopper**
Brukerveiledning
- SV Sughandtaget**
Bruksanvisning
- PT Barra de apoio de sucção**
Manual de instruções

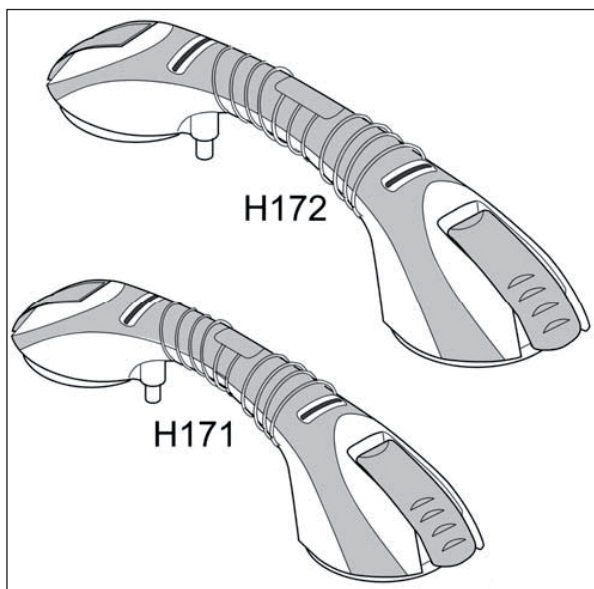
Fig. 1

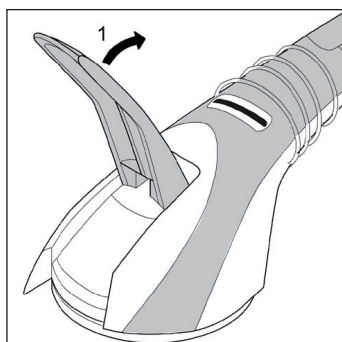
Fig. 2

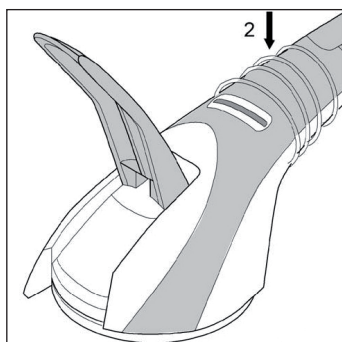
Fig. 3

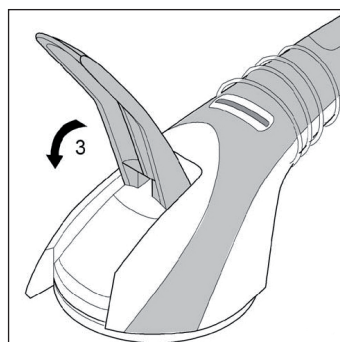
Fig. 4

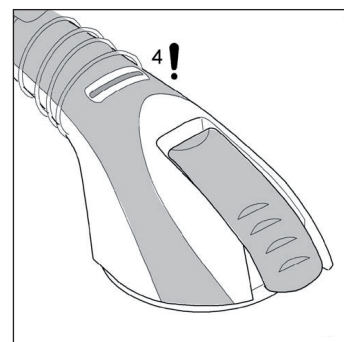
Fig. 5

Fig. 6

Fig. 7

This manual must be given to the user of the product.
Before using the product, read this manual and save it
for future reference!

Yes, you can.

Intended Use

The suction grab bars were designed to support natural movement and not to bear the full body weight of the user. They are intended for short-term use, e.g. on journeys or as a temporary solution after a hospital stay. Do not use for any other purpose.

As the following safety information does not encompass all possible circumstances, it is up to the user in the individual case to check before use whether the suction grab bar is suitable for the intended application, under the existing conditions.

WARNING Risk of falling!

- If the suction grab bar is used for any purpose other than that intended, e.g. as a sports device for "chin-ups" or as a climbing aid, it can suddenly become detached.
- ▶ Only use the suction grab bar for its intended purpose.
 - ▶ Do not use as sports equipment.
 - ▶ Do not use as a climbing aid.

Operating temperature: 10 - 30 °C

Service Life

The expected service life of this product is five years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Safety

WARNING Risk of falling!

- On unsuitable surfaces, the suction grab bar may not achieve the required suction force and may as a result suddenly be detached from the surface.
- ▶ Only use the suction grab bar on smooth, non-porous and firm surfaces, which are clean and dry, steamproof and imperviously sealed.
 - ▶ Never attach a suction grab bar on a joint.

WARNING Risk of falling!

- ▶ Do not use if defective.

WARNING Risk of falling!

- Through ageing, incorrect installation or as a result of a dry construction method, it is possible that the adhesive force of tiles and surface materials may be insufficient. For this reason, the prepared surface may become detached if load is applied to the correctly mounted suction grab bar.
- ▶ Check the stability of the prepared surface before using the suction grab bar for the first time.

WARNING Risk of falling!

- Different ambient conditions can reduce the suction force.
- ▶ Only use the suction grab bar at temperatures below 30 °C and up to a maximum altitude of 600 m above sea level.
 - ▶ Do not expose the suction grab bar to direct sunlight for prolonged periods.
 - ▶ Never attach a suction grab bar to a window.

WARNING Risk of falling!

- Excessive or intermittent loads can lead to spontaneous loss of suction force.
- ▶ Before using the suction grab bar, check the suitability for the relevant user and his/her usage behavior.
 - ▶ The maximum load that can be applied to the suction grab bar is 30 kg.

Scope of delivery

- ▶ Fig. 1.

Assembly

WARNING Risk of falling!

- Incorrect mounting may lead to the suction grab bar becoming suddenly detached from the surface. The check display on the suction grab bar allows the user to check if it is mounted correctly (it is not a vacuum display!).
- ▶ Never use a suction grab bar if the check display is "red".
 - ▶ Ensure that only undamaged, oil, dust and dirt-free suction grab bars are mounted.

- ▶ Fig. 2 - 5.

Usage

WARNING Risk of falling!

- ▶ Always check that the suction grab bar is secure before using and, if in doubt, re-position.

- ▶ Fig. 6, 7.

Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

- ▶ Check product regularly for damage and to ensure that it is securely assembled.

Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

- ▶ Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG002 and your local infection control policy.

- ▶ Always clean the suction grab bar by hand. (No machine washing)
- ▶ Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
- ▶ Max. cleaning temperature 30 °C.

Re-Use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean the product thoroughly and check the safety functions, and reset if necessary.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

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Invacare AB, Fagerstagatan 9,
Aquatec Operations GmbH