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99907 819 19338

User Instructions

Aluminium-Shower stool with Armrests

Item No. 78045 000

Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

The shower stool with armrests provides individuals that suffer from impaired standing stability but with sufficient sitting stability support during their daily hygiene under the shower. The recess in the seat facilitates intimate hygiene and avoids the necessity of standing up and/or bending over during bodily hygiene. The shower stool promotes safety and independence.

⚠ Safety Indications ⚠

- Users heavier than 130 kg may not use the shower stool!
- The foldable shower stool with armrests should only be used, if the individual can sit by himself without any aid. The shower stool may not be used, if you are unable to control your sitting balance!
- Only use the shower stool, if it is in perfect technical condition!
- Before every use please ensure that the shower stool has a stable and secure footing in the shower cabin and that all 4 feet of the stool have been set at the same height! Please ensure that the spring pins have securely locked into place!
- Do not alter the height settings when sitting on the shower stool!
- Do not lean too much forward or backward when sitting on the shower stool to avoid the stool from tilting!
- Do not burden the armrests on one side when sitting down or getting up from the stool to prevent the stool from tilting to one side!
- Do not alter the construction, if not approved by the manufacturer!
- Please observe that in wet shower cabins there is a sliding hazard during sitting down or getting up from the stool – in particular when using cleansing additives or soap!
- Only use the shower stool for its designated use and not as a stepladder substitute or toy!
- Due to the tilting hazard children may not use the shower stool unattended!

Scope of delivery

The contents of delivery includes 1 shower stool frame with armrests, 1 seat, 4 stool legs, 1 user manual, 1 packaging and transport box. Please check the content of the box. In case of visible faults or missing parts, please contact your specialist dealer.

Assembly/Use

To assemble please fit the separate seat onto the frame with the cut-out pointing towards you. There must be a clearly audible click when it engages. Make sure it has a firm fit. (ill. 1-3)

ill. 1


ill. 2

ill. 3

Parts:

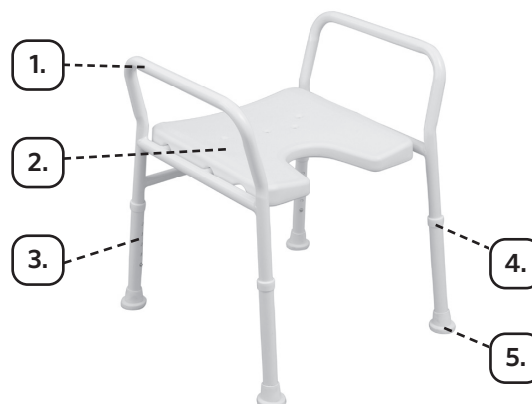1. Armrests
2. Seat
3. Legs of stool
4. Spring pins
5. Non-slide chair leg stoppers

Adjustment of seat height

The shower stool with armrests can be height adjusted via 4 bores in the stool legs by max. 10 cm:

1. Insert the spring pin.
2. Adjust desired length by pulling out or shifting in the stool legs. Ascertain that the bores of the stool legs do match with the bores of the receptacle pipes!
3. Allow the spring pins to engage at the desired position and ascertain that they have securely locked into place. Only if the pins stick out of the bores visibly and noticeably the shower stool has securely been height adjusted and an unwanted telescoping of the pipes can be prevented. (ill. 4)
4. Please ascertain that all 4 stool legs have been adjusted at the same height.

Cleaning/Disinfection

The shower stool should only be used on intact skin. A cleaning agent that is gentler to material should be used for manual cleaning. Only DGHM-/VAH-listed material-friendly surface disinfectant should be used. Please see the relevant manufacturer's instructions for a selection of suitable cleaning agents and disinfectants and their use.

Re-use

The product can be re-used. It must be cleaned and disinfected according to instructions. It must be inspected by specialist shops for any faults and then authorised before it can be used again. Please order our hygiene and maintenance guidelines.

Material

- Seat: PE
- Frame: powder coated aluminium tube
- Chair leg stoppers: rubber

Disposal

Please ask your specialist dealer for the regional disposal regulations.

Technical Details*

- Dimensions seat: 41x40 cm (W x D)
- Total height: 62-72 cm
- Total width: 54 cm
- Total depth: 58 cm
- Seat height: 42-52 cm
- Number of height adjustments: 4
- Height of armrests: 24 cm
- Max. user weight: 130 kg
- Weight: 2,5 kg

* Dimensions of the shower stool. Please allow for slight deviations.

This device complies with Regulation (EU) 2017/745 on medical devices and bears a CE marking.

If a serious incident occurs involving this medical device, contact the manufacturer of this device at info@russka.de as well as the competent authority, the Medicines & Healthcare products Regulatory Agency MHRA, www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency.

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